

## AZIENDA SPECIALIZZATA DA 20 ANNI NEI DISPOSITIVI DI PROTEZIONE INDIVIDUALE Contatti: Tel. 0331/464802 email: <u>info@tekno3.it</u> Sito internet: www.tekno3.it Via Kennedy, 1A

### **SCHEDA TECNICA**

SP400DISPONN - SP200DISPONN (NN: number of cans in the box)

# PRODUCT DESCRIPTION

Ice spray. Cans content: 200 mL and 400 mL



# TECHNICAL AND MATERIAL FEATURES

Ingredients: Isobutane, propane, butane, methylal, isopropyl myristate,

menthol LATEXFREE

The lot is indicated on the top of the can.

Expiration date: 5 years from the date of production. It is shown on the bottle cup

#### CLASSIFICATION

Medical Device Class IIA

Classification according to Annex IX, paragraph III, rule 9 of European Directive 93/42/EEC and subsequent amendments (including Directive 2007/47/EC)

#### **MANUFACTURER**

### INDICATIONS FOR USE/INSTRUCTION FOR USE

Used for relieving pain caused by injuries, scrapes, falls or sprains, and in cases requiring cryotherapy and application of cold. Thanks to its anaesthetic effect induced by cold on the injured area, sports activities can be resumed quickly.

Shake before use. Spray product on the affected area from a distance of at least 20 cm for 3-4 seconds. Repeat application if necessary. The mechanical anaesthesia induced by cold eliminates pain temporarily.

Store at room temperature, away from heat, open flames or sparks.

Keep away from sunlight and do not expose to temperatures exceeding 50°C/122°F.

PCS/BOX							
	12 or 24	LAYERS PALLET					
CARTON SIZE		PALLET SIZE					
CARTON LAYER		PALLET WEIGHT					
CARTON PALLET					Pi	The second	> ###
SYMBOLOGY							
Symbology ac	cording to the norm	JNI CEI EN ISO 15	223-1:2017	C € 0546			
REVISION DATE		APPROVED BY					
01/06/2021 -	- REV. 06	DG					